*A Guide for using Point 4  
Optimized Particle Composite System*

CE  
0086

## *A Guide for using Point 4 Optimized Particle Composite System*

**Thank you** for your decision to purchase another quality Kerr product. For over 100 years, Kerr has built a solid reputation for leading-edge products that represent a great value to the dental community. Please take a moment to read these instructions before using the material.

Point 4 is a light cured resin based composite dental restorative which contains approximately 76% by weight (57% by volume) inorganic filler with an average particle size of 0.4 microns. All shades are radiopaque. Point 4 represents years of research aimed at bridging the gap between microfill and hybrid composite materials. A proprietary grinding process combined with sophisticated rheological modifiers allow Point 4 to polish to a very high, long lasting luster similar to microfills, while providing strength equivalent to today's microhybrids.

It is recommended that the composite be stored at ambient temperature. Shelf life of Point 4 is 3 years from manufacture date. Expiration date is marked on syringes and on Unidose 10-pack boxes. Do not use after expiration date.

It is also recommended that a reliable high power visible light source such as the Kerr Demetron 501 light curing unit be used with this product. Every effort should be made to protect the composite from visible light prior to use.

### **PLEASE NOTE THE FOLLOWING:**

You have purchased this composite in either a bulk syringe or Unidose™ delivery. Either delivery has certain handling and storage requirements that if followed, will provide excellent performance for the user over the life of the material. Please follow the appropriate recommendations for the delivery choice that you have made.

**Syringe Delivery:** To ensure optimal performance of syringes during use: after dispensing the composite, relieve pressure by turning the syringe handle back two turns. This will prevent material build-up in the cap and provide reliable composite use after storage.

**Unidose Delivery:** Unidose tips are designed for **SINGLE PATIENT USE ONLY**. Do not re-cap and/or re-use the Unidose tip once material has been dispensed for that patient.

### **PRIOR TO PLACEMENT: RECOMMENDATIONS ON PROPER BONDING**

- Isolate tooth from adjacent teeth. Use clear thin strips. Rubber dam is highly recommended.
- Moist dentin surfaces – rather than dry, desiccated dentin – are shown to actually enhance the bond strengths of today's modern bonding agents (such as OptiBond Solo Plus). NOTE: "Moist" does not infer blood or saliva contamination.
- Placement sequences, such as those outlined in this instruction booklet, should be followed closely.
- Take a moment to identify the surface to which you are bonding.
- Etching dentin is recommended.
- Please take care to ensure that your air line is free of oil and other contaminants.

### **THE USE OF LINERS**

Some practitioners prefer the use of liners prior to placement of adhesive. In the case of calcium hydroxide, this is not necessary when using Kerr adhesives (OptiBond Solo Plus, OptiBond FL, OptiBond). Unique glass-filled technology and high bond strengths make OptiBond a true replacement for conventional liner materials.

## **OPTIBOND SOLO PLUS**

If you have purchased Point 4 in kit form, you will notice that OptiBond Solo Plus is included in the system. This is the recommended adhesive for use with all Kerr composites. OptiBond Solo Plus is 15% filled with the same 0.4 micron size barium glass found in the Kerr Point 4 composite. This unique technology provides the highest level of protection against microleakage, while sustaining high bond strengths to a variety of surfaces. The filler not only reinforces the hybrid zone, but penetrates the dentin tubules as well, creating a true “structural bond” not found in unfilled or even “nano” filled adhesive systems. With OptiBond Solo Plus, adhesion promoters are carried in an ethanol solvent, thus greatly reducing the tedious need for multiple coats and constant re-application commonly found with acetone adhesives. OptiBond Solo Plus is indicated for both direct and indirect (in conjunction with a luting agent) bonding applications.

OptiBond Solo Plus is available in two delivery systems: Bottle and Unidose™. Kerr’s Unidose delivery adhesive system was the first of its kind, designed as a progressive step toward ensuring fresh material for each bonding procedure as well as the ultimate in infection control. It is protected under U.S. Patent 5,860,806.

### **DIRECT BONDING POINT 4 COMPOSITE TO ENAMEL/DENTIN WITH OPTIBOND SOLO PLUS**

1. Isolate and dry.
2. Place Kerr Gel Etchant\* on enamel and dentin for 15 seconds. Rinse thoroughly, ensuring that all etch is removed.
3. Dry lightly, do not desiccate.
4. Apply OptiBond Solo Plus to enamel/dentin surface with light brushing motion for 15 seconds.
5. LIGHTLY air thin for maximum 3 seconds.
6. Light cure for 20 seconds.\*\*
7. Proceed with placement of Point 4 composite.

Note: Kerr Disposable Mixing Wells, Gel Etchant tips and Applicator Tips are designed for single use only.

### **PLACEMENT OF POINT 4 COMPOSITE**

Point 4 Optimized Particle Composite is available in a wide variety of shades. For details on all available shades and their various uses, please refer to “Explanation of Point 4 Shading System” below for more information.

1. Select the desired shade(s).
2. Syringe Delivery: after extruding material out on pad, make sure to back off the syringe at least two full turns to relieve pressure on the material.  
Unidose Delivery: insert tip and properly align within dispenser. Place Unidose tip at the deepest portion of the preparation. Caution: extrude slowly with even pressure.
3. Increments should be no more than 2mm at a time. After placing an increment, stroke the composite to ensure marginal adaptation.
4. Light cure each increment for 40 seconds. When multiple surfaces are available, cure each surface for this recommended time.\*\*

### **EXPLANATION OF POINT 4 SHADING SYSTEM**

Point 4 uses the VITA shading system for its base and opaque shades. Furthermore, there are six auxiliary shades available as well (three translucent shades, and three extra light shades).

### *Translucent Shades (T1, T2, T3)*

The three translucent shades available with Point 4 allow you to create highly esthetic restorations with relative ease. Designed to be placed in very thin layers over one of the base VITA shades, these specialty shades give the dentist everything he/she needs to virtually replace tooth structure with lifelike results. With a little practice, it is possible to build true depth and beauty to every restoration. A brief description of the 3 translucent shades:

T1 – a clear translucency, designed to bring lifelike depth to your restorations without modifying the base VITA shade(s) used below it.

T2 – a slightly yellow translucency.

T3 – a slightly gray translucency.

### *Extra Light Shades (XL1, XL2, XL3)*

The extra light shades can be useful to match bleached as well as deciduous teeth. All three of them are lighter than the lightest VITA shade (B1), with the lightest (or whitest) shade being XL3. In order to become familiar with these three shades some experimentation outside of the mouth is recommended. The available shade guide provides cured examples of each XL shade for your convenience as well.

## **AVAILABLE SHADES**

The following is a listing of the shades available with the Point 4 Optimized Particle Size Composite System:

Base Shades		Translucent Shades		Extra Light Shades
A1	B4	T1		XL1
A2	C1	T2		XL2
A3	C2	T3		XL3
A3.5	C3	Opaque Shades		
A4	C4	A2	B1	
B1	D2	A3	B2	
B2	D3	A3.5	C2	
B3	D4	A4	D2	

Syringes have the shade stamped on the top of the plunger, near the paddle. Also, the shade is indicated on the label of each syringe. Unidose tips are stamped with the shade as well, to eliminate any confusion should the original boxes become misplaced.

Unidose Composite Gun Sterilization guidelines are as follows: Steam Autoclave: 30 minutes @ 270 °F.

## **OPTIGUARD**

OptiGuard is a composite sealant that can be used as a final overlay on top of all composite restorations. Modern high-speed finishing materials may create tiny cracks that can lead to premature composite wear, as well as continued propagation below the surface. Applying OptiGuard on the composite surface may increase the restoration's chances for extended wear and durability.

### *Directions for Use-OptiGuard*

1. Rinse final restoration and dry
2. Using Kerr Gel Etchant\*, etch the restorative surface and approximately 2mm of enamel beyond the margin for 15 seconds.

3. Rinse tooth thoroughly, and dry with oil-free air.
4. After dispensing a small amount of OptiGuard in mixing well, apply with Kerr Applicator to etched surface in a thin layer. Blow lightly to even the application on the surface.
5. Light cure the surface for 20 seconds. \*\*
6. Check occlusion when necessary

**\*Caution:** contains phosphoric acid. Avoid contact with skin, eyes, and soft tissue. In case of contact with skin or eyes, flush immediately with water, get medical attention for eyes. Do not take internally.

**\*\* Recommended Cure Times:** L.E.Demetron I, 10 seconds or Optilux 501 in Boost mode 10 seconds, Ramp Mode 20 seconds or Regular Mode 20 seconds. For all other lights see manufacturer's recommendation.

**CAUTION:** Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

### **Limited Warranty-Limitation of Kerr's Liability**

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists using Kerr's product. Such advice does not expand Kerr's limited warranty or relieve the dentist of testing Kerr's products to determine their suitability for the intended uses and procedures. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY OR FITNESS FOR A PARTICULAR PURPOSE.

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Kerr U.K. Ltd.
Kerr Australia Pty., Ltd.
Sybron Dental Specialists Japan, Inc.